

SAFETY PRECAUTIONS

- Never leave the user unattended.
- Always use on a level surface and ensure all feet are in contact with the ground.
- Do not exceed maximum weight limit (100kg).
- Do not lean too far or over extend while sitting on the chair.
- Do not stand on the chair.
- Inspect at regular intervals for any wear or damage.
- Ensure that all hardware are tight and secure before use.

CLEANING INSTRUCTIONS

1. Clean with a disinfectant surface cleaner using a soft cloth and wipe dry.
2. Ensure product is dry before storing.
3. Store in a cool dry place.

Note: As the product is steel, it is important to keep dry when not in use to prevent rust.

FEATURES

- Powder coated steel frame
- Height adjustable
- Removable backrest
- Plastic armrests
- Rubber tips for grip
- Plastic seat with lid
- Collapsible pail with lid
- Compact packaging (some assembly required)
- Conditional warranty applies


SPECIFICATIONS

Seat height:	42 – 52cm
Seat width:	33cm
Seat depth:	40cm
Seating width:	45cm
Seating depth:	42cm
Backrest height:	22cm
Armrest height:	19cm
Overall width:	54.5cm
Overall length:	55cm
Weight capacity:	100kg
Net weight:	5.7kg

Standard Steel Commode




PARTS LIST

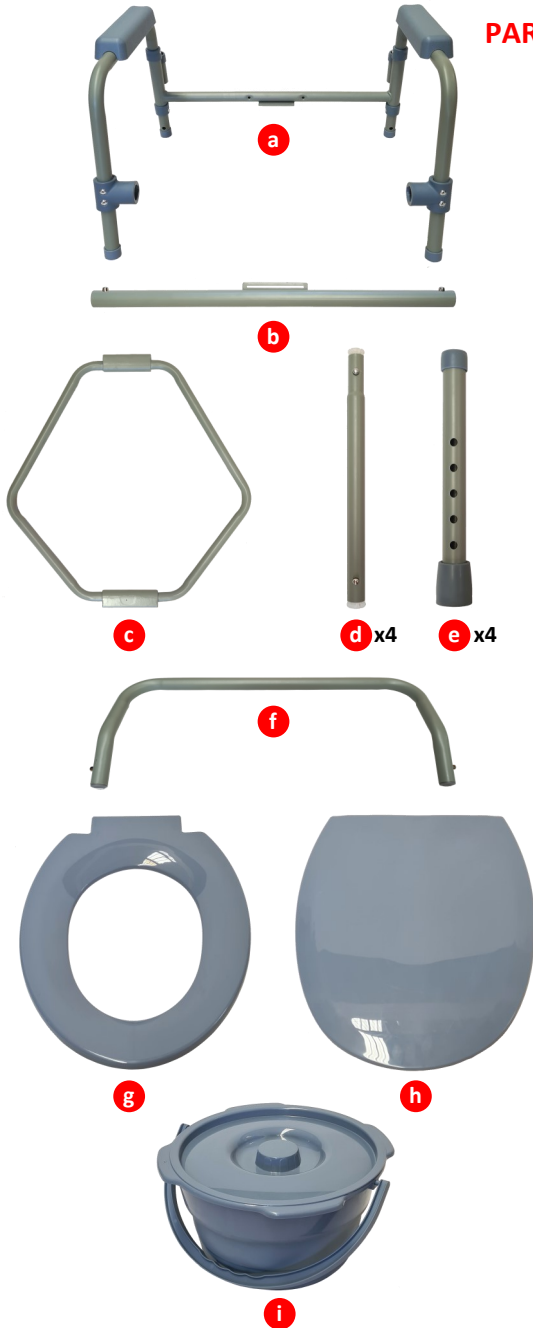


ASSEMBLY (no tools required)


1. Attach crossbar (b) to frame (a). Lock in securely with spring buttons

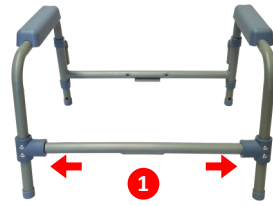

2. Insert pail holder (c) into slot holes with the wider section towards the front end.

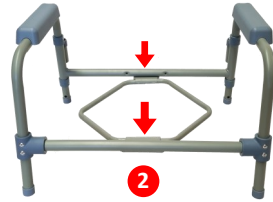


3. Attach backrest (f). Insert ends into brackets and lock in place with spring buttons.

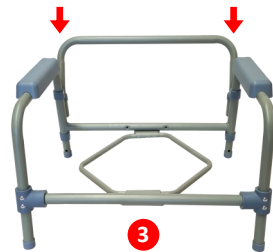


4. Assemble legs (x4), insert top (d) into the bottom (e) section.


Attach legs to frame.

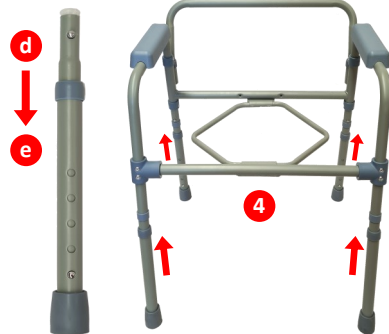

5. Snap on toilet seat (g) centred onto frame.

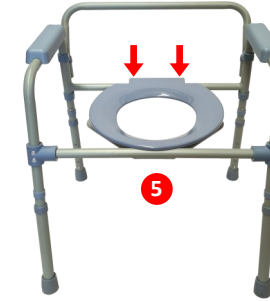

6. Snap on toilet lid (h) centred onto frame.

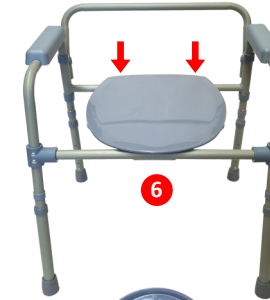

7. Insert waste pail (i) in its holder (c).



8. Adjust leg length to a suitable seating height.

Ensure all legs are at the same height setting.

