


REAR CASTOR BRAKE

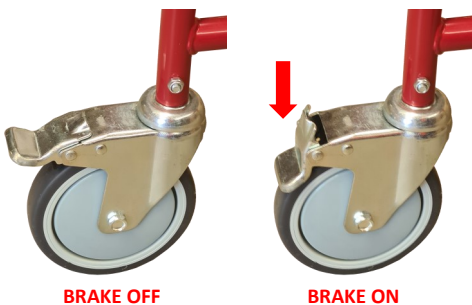

BRAKE OFF

BRAKE ON



SAFETY PRECAUTIONS

- Never leave the user unattended.
- Ensure brakes on rear castors are applied before use.
- Always use on a level surface and all wheels are in contact with the ground
- Do not exceed the maximum user weight limit (100kg)
- Do not stand on the chair.
- Do not stand on footrests while getting on or off chair.
- Inspect at regular intervals for any wear and damage.
- Check all hardware are tight and secure before use.

CLEANING INSTRUCTIONS

1. Clean with a disinfectant surface cleaner using a soft cloth and wipe dry.
2. Ensure product is dry before storing.

Note: As the product is steel, it is important to keep dry when not in use to prevent rust. Using a spray lubricant will assist in moisture displacement and protect from corrosion.

SPECIFICATIONS

Seat Height:	50.5cm
Seat Width:	43cm
Seat Depth:	43cm
Seating Width:	44.5cm
Seating Depth:	48cm
Backrest Height:	37cm
Armrest Height:	23cm
Overall Width:	54cm
Overall Length:	93cm
Weight Capacity:	100kg
Net Weight:	11.9kg

Mobile Steel Shower Chair Commode



PARTS



ASSEMBLING THE FRAME

1. FRAME & CASTORS (tools required)
Connect the **CROSSBARS** to the front and rear of **SIDE FRAMES** using 4 **LONG BOLTS & NUTS**.



Attach the 4 **SWIVEL CASTORS** to the frame legs with 4 **SHORT BOLTS & NUTS**.
BRAKE CASTORS installed to the rear.

2. ARMRESTS

Insert the left and right **ARMRESTS** into the corner tube slots of the **FRAME** and secure with spring button.



3. BACKREST

Insert backrest into seat frame and secure with C Clips.



4. COMMODE SEAT

Position commode seat on frame and click into place.



5. SWING AWAY FOOTRESTS

Attach swing away footrests and ankle strap to complete assembly.



COMPLETE

