

## CLEANING INSTRUCTIONS

The shower chair can be disinfected by spraying it with any standard non-abrasive hard surface cleaner.

1. Gently wipe the disinfectant onto the surface using a soft cloth and let sit for several minutes.
2. Rinse the shower seat to make sure all disinfectant has been removed.
3. Wipe dry before storing.

## Shower Chair not level in the shower? HOW TO FIX

Rotate the feet to accommodate the contour or shape of the tub and shower, making sure they are flat to the surface. Ensure the tension buttons on each leg are positioned evenly so the chair is completely level.

## SPECIFICATIONS

Seat Height:	45.5 - 55cm
Seat Width:	40cm
Seat Depth:	33cm
Seating Width:	50cm
Seating Depth:	37cm
Backrest Height:	33.5cm
Overall Width:	60cm
Overall Depth:	54cm
Weight Capacity:	150kg
Net Weight:	6.4kg

## IMPORTANT SAFETY INFORMATION BEFORE USE

It is the user's responsibility to ensure the chair is suitable for use.

- Do not exceed the maximum user weight limit of 150kg.
- Always ensure that the legs are evenly adjusted so the chair is level and stable.
- Inspect at regular intervals for wear and damage. Ensure the plastic parts and rubber feet are in good condition; replace rubber feet if damaged.
- Do not stand on the Swivel Shower Chair.
- Never leave the user unattended.
- Not recommended for use with individuals who are severely mentally, cognitively, physically or vision impaired.

**NOTE:** Failure to follow the above noted safety instructions can lead to injuries.

# The RITZ range

## RTZ-B-100 Deluxe Swivel Chair




## Step 1

Remove parts from box and plastic bags.

### Parts include:

- A** Seat x1    **B** Right Armrest x1    **C** Left Armrest x1  
**D** Backrest x1    **E** Legs x4

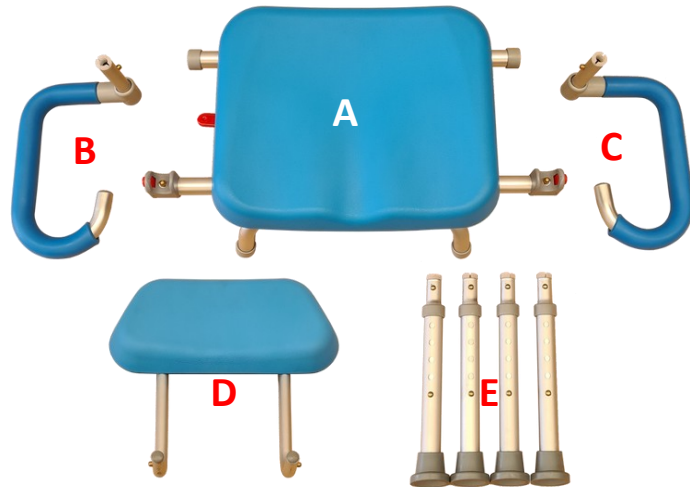

## Step 2

Align **Leg (E)** to leg connections on **Seat Section (A)** and press tension button to insert the leg and slide until the tension button locks in place. Repeat step for the other three legs and align evenly. Adjust the rubber feet by rotating them so they are level to the floor.



## Step 3

Align **Backrest (D)** tubes to rear tubes on **Seat Section (A)**. Press tension buttons on the backrest tubes to insert the backrest into the rear seat tubes and slide until the tension buttons lock in place.



## Step 4

(a) Align **Armrest (B) & (C)** to side tubes on **Seat Section (A)** and push tension button to slide armrest until the button locks in place. Repeat for other side.

**The Swivel Shower Chair is now ready for use.**

(b) Swing down armrests and lock into place when in use. Press in red button to release armrest and lift up for ease of use when sitting / exiting from the side.



## OPERATION

Press and hold down the red lever on the side to rotate the seat; release lever to lock into place every 90 degrees.

Adjust the height of the Swivel Shower Chair to the most comfortable position. Ensure all four legs are aligned to the same height to provide a level base. The rubber feet can be rotated so the chair is level with the floor.

