

CLEANING INSTRUCTIONS

The shower chair can be disinfected by spraying it with any standard non-abrasive hard surface cleaner.

1. Gently wipe the disinfectant onto the surface using a soft cloth and let sit for several minutes.
2. Rinse the shower seat to make sure all disinfectant has been removed.
3. Wipe dry before storing.

Not sliding easily? HOW TO FIX:

Please note that a light lubricant is applied to the horizontal rails that the chair slides on. This lubricant is applied at the factory before the item is shipped out. Sometimes this lubricant can evaporate if high temperatures occur during the shipping process. If your chair is not sliding easily, apply a light coating of LUBRICANT (silicone base recommended) to both horizontal rails.

SPECIFICATIONS

Seat Height:	14cm (from support base)
Seat Width:	40cm
Seat Depth:	32.5cm
Seating Width:	54cm
Seating Depth:	36cm
Backrest Height:	29cm
Armrest Height:	15cm
Fits Tub Width:	56 - 71cm (2.5cm increments)
Overall Width:	70.5 - 85.5cm
Overall Depth:	55.5cm
Weight Capacity:	150kg
Net Weight:	8.25kg

IMPORTANT SAFETY INFORMATION BEFORE USE

It is the user's responsibility to ensure the chair is suitable for use.

- Do not exceed the maximum user weight limit of 150kg.
- Ensure the support bars are secure against the bath walls; frame must be fixed in place and immovable
- Inspect at regular intervals for wear or structural damage to frame.
- Do not stand on the bath seat.
- Never leave the user unattended.
- Not recommended for use with individuals who are severely mentally, cognitively, physically or vision impaired.

NOTE: Failure to follow the above noted safety instructions can lead to injuries.

The RITZ range

RTZ-B-400 Padded Rotating Sliding Bath Seat




Step 1

Remove parts from box and plastic wrapping.

Parts include:

- A** Backrest
- B** Seat Base Component
- C** Side Support Arm
- D** 2x Seat Armrests
- E** 2x Side Supports
- F** Screw and Bolt Parts

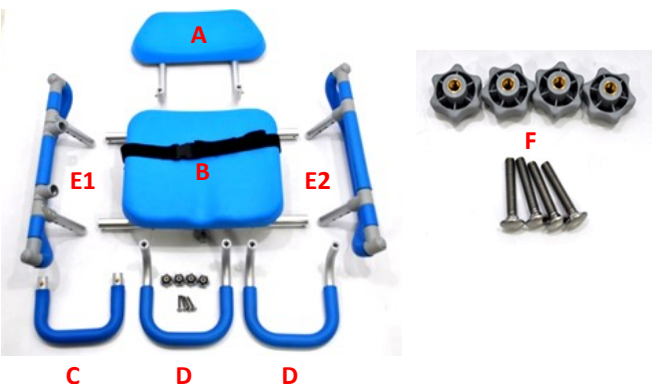


Step 2

Position both **Side Supports (E1 & E2)** to either side of the **Seat Base (B)**; and insert into each side of the sliding tubes underneath the **Seat Base (B)**.


Step 3

Extend the supports to fit the width of the bath tub, insert the **Bolts (F)** (with the plastic knobs on the inside) to the closest appropriate holes and tighten.

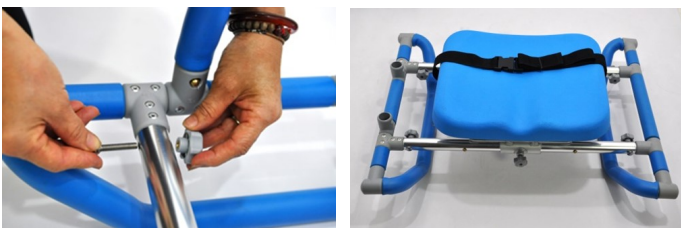

Step 4

Attach each **Armrest (D)** onto the **Seat Base (B)**.
Make sure all tabs lock into place.

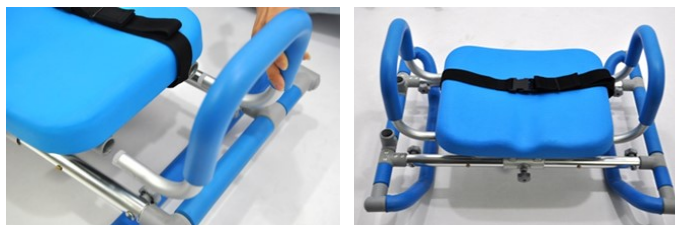

Step 5

Insert the **Side Support Arm (C)** into **Side Support (E1)**.
Make sure all tabs lock into place.



Step 6

Insert the **Backrest (A)** into the rear of the **Seat Base (B)** and click into place.

The chair is now ready to be placed in a standard bath.



OPERATION

The seat has an integrated swivel and sliding mechanism that will assist the caregiver in rotating the client over the tub wall with minimal effort.

1. The width of the swivel chair should be securely supported on each side by the bath walls and stable. The support arm should be situated away (opposite) from the loading side.
2. Pressing down on the red lever located the side of the seat allows rotation of the seat. The chair locks into place every 90 degrees.
3. Pressing the grey button located on the sliding rail allows the seat to slide along the rail and locks in 2 positions - the loading side and centre. If necessary, the 3rd position closest to the support arm can also be used by removing the support arm or seat armrest.

Seating

1. Position seat to edge of bath facing outwards.
2. Open the seat belt and adjust the length of the belt. Sit the client down onto the swivel chair.
3. Once seated comfortably, buckle the client into place.
Note: The adjustable lap strap must be worn during the transfer and showering process.
4. Press the grey button to slide the chair to the centre of the bathtub until it locks into place.
5. Gently raise the client's legs and pivot them over the side and into the tub.

Exiting the seat

1. Press the red lever on the side of the swivel seat
2. Gently raise the client's legs and pivot them over the side of the tub
3. Press the grey button located on the sliding rail to release lock and slide the seat towards the edge; lock into position.
4. Unbuckle the seatbelt and exit from the bath seat.