

DESCRIPTION

Over Toilet Frames are designed to increase the height of the toilet seat and includes arm supports to assist the user in transferring onto and off the toilet safely.

Includes splashguard.

SPECIFICATIONS

Seat height range: 43.5-58.5cm

Width between arms: 47.5cm

Armrest height: 22.5cm

Seating width: 43.5cm

Seating depth: 42cm

Width at base: 62cm

Depth at base: 67.5cm

Unit weight: 3.9kg

Weight capacity: 200kg

200kg Bariatric Over Toilet Aid



SETTING UP

ADJUST THE HEIGHT OF THE LEGS

On each leg, press the spring button and adjust the leg piece to the desired height making sure the toilet aid sits level.

Ensure that the height adjustment spring buttons are fully engaged prior to use.

PLACEMENT

Lift the toilet lid and seat.

With the hinged side to the rear, place the over toilet frame so that the seat lines up with the surround of the toilet bowl.

If using a splashguard, lift the seat of the over toilet frame and place the splashguard on the supports. Ensure the splashguard does not touch the toilet bowl. The height of the over toilet frame may need to be increased to stop the splashguard from touching the toilet bowl.

HOW TO USE

Stand with the back of your legs against the over toilet frame.

Use both hands on the arms of the over toilet aid when lowering onto, and rising up from the toilet seat.

Important: To transfer on and off safely, ensure equal force is applied to both armrests to avoid the toilet frame from tipping.

SAFETY

- Avoid tilting or leaning over too far.
- Use caution when having the bench at higher settings as it may become unstable, there is a greater risk of tipping backwards if the user sits down heavily.
- Ensure that the toilet paper is easily reached.
- Be sure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities. If you are uncertain, seek the advice of a qualified therapist or carer.
- Care must be taken to ensure stability and all feet are positioned firmly on the floor.
- Remove all mats and other possible tripping hazards. These increase the risk of falling.
- Do not stand on the toilet aid.

MAINTENANCE AND CLEANING

- Before use, always check to ensure the over toilet aid is in good working condition.
- DO NOT USE if product shows signs of damage, corrosion or rust
- Regularly clean the over toilet aid with a mild disinfectant and wipe dry. Do not use abrasive cleaning agents or solvents
- Dry product completely after use and before storing to prevent corrosion
- Store in a dry place