

Ideal for people who are unable to stand for long periods of time. Shower chairs enable a person to bathe safely while remaining seated. It gives the user ongoing comfort, stability and greatly reduces the chance of slipping and falling.

SPECIFICATIONS

Seat height range:	41.5 - 56.5cm
Width between arms:	56cm
Armrest height:	29cm
Seating width:	56cm
Seating depth:	41cm
Width at base:	74cm
Depth at base:	51cm
Net weight:	4.5kg
Weight capacity:	250kg

Bariatric Shower Chair



SAFETY

- Before use, always check to ensure the product is in good working condition.
- Be sure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities. If you are uncertain, seek the advice of a suitably qualified therapist or carer.
- With all products, care must be taken to ensure stability and all feet are positioned firmly on the surface / floor.
- Avoid tilting or leaning over too far.
- DO NOT stand on the chair.

SETTING UP

ADJUST THE HEIGHT OF THE LEGS

Adjust the chair to an appropriate seating height ensuring your feet are touching the floor.

- On each leg, press the spring button and adjust the leg piece to the desired height making sure the shower chair sits level.
- **IMPORTANT:** Ensure that the spring button is fully engaged in the adjustment hole and the leg does not move

MAINTENANCE AND CLEANING

- Check condition regularly for wear and tear.
- DO NOT use if the product shows any sign of damage, corrosion or rust.
- Clean the frame and seat with mild soap / detergent and water; wipe dry.
- Do not use abrasive cleaning agents or solvents.
- To prevent rust - Dry product completely after use and before storing.
- Spray lubricants will assist in water/moisture displacement and protect from the product from corrosion.
- Store in a dry place when not in use.

