

DESCRIPTION

Ideal for people who are unable to stand for long periods of time.

Shower chairs enable a person to bathe safely while remaining seated. It gives the user ongoing comfort, stability and greatly reduces the chance of slipping and falling.

SPECIFICATIONS

Seat height range:	42 - 57cm
Width between arms:	49cm
Armrest height:	26cm
Seating width:	45cm
Seating depth:	41cm
Width at base:	62cm
Depth at base:	63cm
Net weight:	4.5kg
Weight capacity:	200kg

200kg Bariatric Shower Chair



SETTING UP

ADJUST THE HEIGHT OF THE LEGS

On each leg, press the spring button and slide the leg piece to the desired height.

Adjust each leg so the bench sits level. Adjust the chair to an appropriate seating height to allow the user to sit and stand with ease.

IMPORTANT: Ensure that the spring buttons have fully engaged the notch and the leg does not move.

MAINTENANCE AND CLEANING

- Check condition regularly for wear and tear.
- DO NOT use if the product shows any sign of damage, corrosion or rust.
- Clean the seat and frame with a mild disinfectant.
- DO NOT use abrasive scrubbers or cleaning agents.
- To prevent rust - dry product completely after use and before storing.
- Store in a dry place when not in use.

SAFETY

- Before use, check the product is in good working condition.
- Be sure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities. If you are uncertain, seek the advice of a suitably qualified therapist or carer.
- With all products, care must be taken to ensure stability and all feet are positioned firmly on the surface / floor.
- Avoid tilting or leaning over too far.
- Ensure that items required for bathing are easy to reach when seated.
- DO NOT stand on the chair.
- **WARNING:** Use caution when having the bench at higher settings as it may become unstable, there is a greater risk of tipping backwards if the user sits down heavily.