



1. Height adjustable handles
2. Locking handbrake
3. Removable back rest
4. Height adjustment knob
5. Back rest attachment bracket
6. Lift up seat
7. Under seat storage bag
8. Folding bracket
9. Rear wheel with brake
10. Swivel front wheel

HOW TO USE

UNFOLDING THE ROLLATOR

1. Spread the front and rear legs apart;
2. Then push down on the folding bracket (8) on each side to full extension.

BRAKES

Caution: Always check brakes before using rollator.

Pull the brake lever (2) – up to brake and down to lock the brakes, pull upwards to disengage.

Keep the brake handles free from dirt, dust and grime. Build-up of particles could cause your brakes not to engage properly. If the brakes are not engaging, or contacting the wheel while not being applied, adjustments can be made at the adjuster screw before the brake handle and / or at the foot of the cable near the brake pad (tools may be required).

HANDLEBARS – INSTALL / ADJUST HEIGHT

Insert handle into tube of the frame.

Adjust position to required height; locate nearest hole and insert bolt (4). Ensure the bolt head recesses into the 6 side hole in frame; slide washer on with the curved side facing the frame; then screw on knob and tighten. Repeat for other side.

BACK REST – INSERT / REMOVAL

Insert ends of the foam padded back rest (3) into brackets (5) until the buttons lock into holes.

To release, push the buttons in on both sides and pull out.

FOLDING THE ROLLATOR

Lift the seat open; using the strap handle inside, pull the centre bar upward and push the rollator frame together to fold.

Caution: Be careful that fingers or any objects do not get caught between the folding bracket (8) when closing.

SAFETY TIP

Lock handbrakes when sitting down, sitting, and rising off the seat; and where possible, lean the front castors (backrest side) against a solid vertical surface (eg kerb or wall) to prevent the rollator from sliding back when standing up from the rollator.

GENERAL CARE & MAINTENANCE

Always check product is in good working condition before use. Check regularly for wear and tear.

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities.

Seek the advice of a suitably trained specialist.

Care must be taken to ensure stability

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Backrest must be attached to walker prior to use.
- Do not hang objects on the handles or backrest – this could cause the rollator to tip over.

DO NOT use if rollator shows signs of damage, corrosion or rust

- Regular servicing is advised.
- Periodically check that bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- To prevent rust - Do not submerge the product in water and make sure to dry product completely before storing.
- Lubricate moving parts.
- A spray lubricant will assist in water/moisture displacement and protect hardware from corrosion.
- Store in a dry place.

SPECIFICATIONS

Handle height (lowest - highest):	75.5cm - 89cm
Width between handgrips:	45.5cm
Seat height:	44cm
Seat (width x depth):	45 x 30cm
Thickness of seat pad:	3.3cm
Overall depth:	62cm
Overall width:	61cm
Wheel size:	15cm (6")
Weight capacity:	136kg
Unit weight:	5.7kg