

Wide Aluminium Shower Chair



GENERAL CARE & MAINTENANCE

DO NOT use if the frame shows signs of damage, corrosion or rust.

- Before use, always check product is in good working condition.
- Seek advice of a suitably qualified therapist.
- Ensure the user's weight is within the weight capacity of the unit.
- Care must be taken to ensure stability.
- Avoid tilting or leaning over too far.
- Periodically check all hardware and the height adjustment buttons are secure.

CLEANING

- Use warm soapy water and / or a mild disinfectant cleaner only.
- Do not use any abrasive cleaners or materials.
- Dry product completely before storing to prevent rust.
- Store in a dry place.

DESCRIPTION

Ideal for people who are unable to stand for long periods of time. Shower chairs enable a person to bathe safely while remaining seated. It gives the user ongoing comfort, stability and greatly reduces the chance of slipping and falling.

FEATURES

- Textured powdercoated aluminium frame.
- Height adjustable.
- Rubber suction tips for added safety.

SPECIFICATIONS

Seat height	37.5 - 52.5cm
Backrest height:	33cm
Seat Width:	44cm
Width between arms:	53.5cm
Width at base:	62.5cm
Depth at base:	55.5cm
Weight capacity:	140kg
Net weight:	3.10kg
Replacement tip:	RTIP202



ASSEMBLY

1. Place the frame on its side (**Figure 1**).
2. Insert the four screws into each hole (**a**) with a washer on each side of the frame (2 per screw). (See **Figure 2**).
3. Position the seat to align with the screws and while holding the seat in place, partially fasten the screws into the corresponding holes on the seat, do not fully tighten until all four screws are in place (**Figure 3**).
4. Tighten all screws.
5. Attach the four legs (**Figure 4**) to a suitable seating height. Feet should be flat on the floor. Ensure the height buttons have fully engaged.

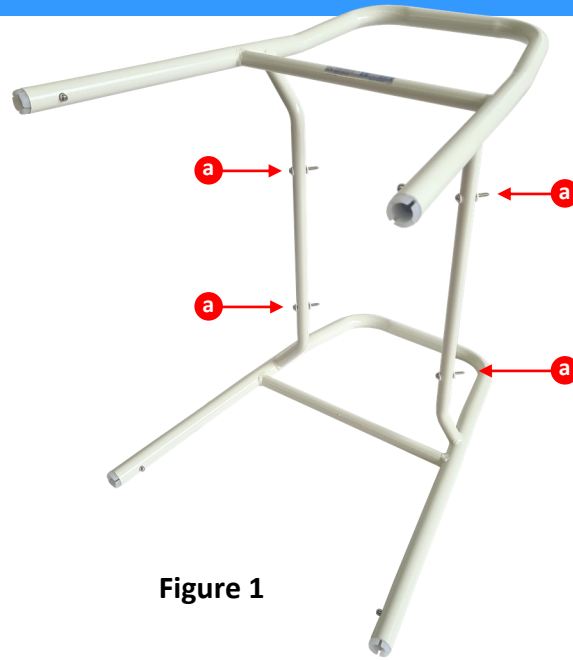


Figure 1



Figure 2

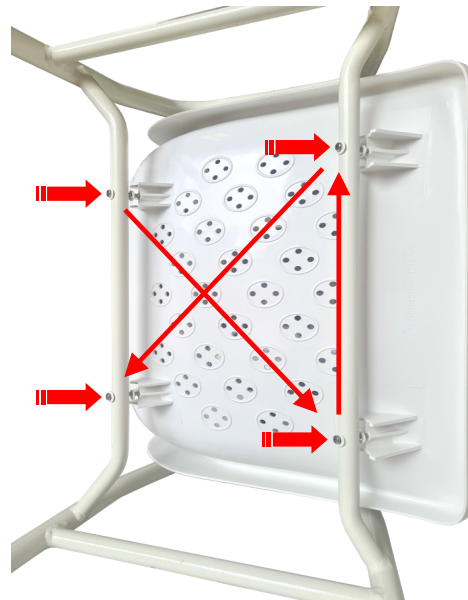


Figure 3



Figure 4

