








WARNINGS

To ensure your safety while using this shower chair, all instructions must be followed.

-  The maximum load capacity of this shower chair is 130kg.
-  Do not use this shower chair if it is obviously damaged or if any parts are missing.
-  Ensure all fixed parts are correctly and firmly attached.
-  This shower chair should be placed indoors in a dry, ventilated, stable area free of corrosive substances.
-  Clean regularly, wiping down with a soft cloth and a mild detergent and/or disinfectant solution.
-  Do not use corrosive solutions and/or abrasive cleaners.
-  Use caution when bench is set at a higher setting as it may be unstable, there is a greater risk of tipping backwards if the user sits down heavily.

Rotating & Sliding Transfer Bath Bench



PARTS DESCRIPTION

1. Backrest
2. Flip up armrests
3. Armrest latch
4. Safety belt
5. Rotating sliding seat
6. Rotate lock button
7. Slide lock button
8. Height adjustable legs
9. Suction cup feet



ASSEMBLY Step 1

Unbox the shower chair on a level floor.



HOW TO SLIDE SEAT

Press the button below the seat (7) and glide along the rail then release.



HOW TO ROTATE SEAT

Pull out the knob under the seat (6), rotate seat then release.



ASSEMBLY Step 5

Attach the back end of the armrest into the tube under the seat by pressing in the locking button and insert armrest until it locks in place. Pinch clip on the front and insert into its bracket. Repeat for other side.



ASSEMBLY Step 2

Press the spring button of the leg tube and insert it into the main frame. Repeat for the other side.



ASSEMBLY Step 3

Press the spring button of the leg extension and insert it into the leg tube. Repeat for the other three leg extensions. The two suction tips should be installed on the side that will be placed inside of the bathtub. Once assembly is complete, adjust the lengths accordingly for a level seating surface over the bathtub.



ASSEMBLY Step 4

Press the spring button of the back brace bar and insert it into the back of the seat.

