

PA330 / PA331 / PAB800 ROLLATOR



PA330

Handle Height:	79 - 91cm
Seat Height:	55cm
Seat Width:	35cm
Seat Depth:	32cm
Backrest Height:	27cm
Overall Length:	61cm
Overall Width:	61cm
Folded Width:	42cm
Between Handgrips:	45cm
Wheel Size:	15 x 2.5cm
Weight Capacity:	110kg
Net Weight:	6.7kg

6" Castor



PA331

Handle Height:	78 - 90cm
Seat Height:	55cm
Seat Width:	35cm
Seat Depth:	32cm
Backrest Height:	26.5cm
Overall Length:	69cm
Overall Width:	61cm
Folded Width:	42cm
Between Handgrips:	45cm
Wheel Size:	20 x 2.5cm
Weight Capacity:	110kg
Net Weight:	6.8kg

8" Castor



PAB800

Handle Height:	77 - 92cm
Seat Height:	54cm
Seat Width:	46cm
Seat Depth:	33cm
Backrest Height:	26cm
Overall Length:	69cm
Overall Width:	74cm
Folded Width:	42cm
Between Handgrips:	52cm
Wheel Size:	20 x 2.5cm
Weight Capacity:	180kg
Net Weight:	11.3kg

Heavy Duty



PA330 / PA331 / PAB800 ROLLATOR



1. Height adjustable handles
2. Handbrake
3. Height adjustment knob
4. Removable back rest
5. Back rest bracket attachment
6. Lift up seat
7. Under seat storage bag
8. Folding bracket
9. Rear wheel with brake
10. Swivel front wheel

HOW TO USE

UNFOLDING THE ROLLATOR

Spread apart the front and rear legs; push down on the folding bracket (8) on each side to full extension.

BRAKES

Caution: Always check brakes before using rollator.

Pull the brake lever (2) – up to brake and down to lock the brakes, pull upwards to disengage

Keep the brake handles free from dirt, dust and grime. Build-up of particles could cause your brakes not to engage properly. If the brakes are not engaging, or contacting the wheel while not being applied, adjustments can be made at the adjuster screw before the brake handle and / or at the foot of the cable near the brake pad (tools may be required).

HANDLEBARS – INSTALL / ADJUST HEIGHT

Insert handle into frame tube.

Adjust position to required height; locate nearest hole and insert bolt (3). Ensure the bolt head recesses into the 6 side hole in frame; slide washer on with the curved side facing the frame; then screw on knob and tighten. Repeat for other side.

BACK REST – INSERT / REMOVAL

Insert ends of the foam padded back rest (4) into brackets (5) until the buttons lock into holes.

To release, push the buttons in on both sides and pull out

FOLDING THE ROLLATOR

Lift the seat open, using the strap handle inside the storage bag, pull up from handle strap inside bag and push the rollator frame together to fold

Caution: Be careful that fingers or any objects do not get caught between the folding bracket (8) when closing

GENERAL CARE & MAINTENANCE

Before use, always check product is in good working condition. Check regularly for wear and tear

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities.

Seek the advice of a suitably trained therapist or mobility specialist.

Care must be taken to ensure stability:

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Backrest must be attached to walker prior to use.
- Do not hang objects on the handles or backrest – this could cause the rollator to tip over.

DO NOT use if the rollator shows signs of damage, corrosion or rust

- Regular servicing is advised.
- Periodically check that bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- To prevent rust - Do not submerge the product in water and make sure to dry product completely before storing.
- Lubricate moving parts.
- A spray lubricant will assist in water/moisture displacement and protect from corrosion
- Store in a dry place.

