

## HAND BRAKE OPERATION

**Braking:** Squeeze brakes (2) gently to reduce speed and to stop.

**Parking:** Push brake handles (2) forward to activate park brake (a faint click will be heard when engaged). Pull back on brakes to release.



## GENERAL CARE & MAINTENANCE

Before use, always check product is in good working condition. Check regularly for wear and tear

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities. Seek the advice of a suitably trained therapist or carer.

### Care must be taken to ensure stability:

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Backrest must be attached to walker prior to use.

### DO NOT use if the rollator shows signs of damage.

- Periodically check that bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- To prevent rust - Do not submerge the product in water and make sure to dry product completely before storing.
- Lubricate moving parts.
- A spray lubricant will assist in water/moisture displacement and protect from corrosion
- Store in a dry place.

## FEATURES

- Liquid coated steel frame
- Seat height adjustable – 4 settings
- Adjustable gutter arm height – 6 settings
- Adjustable gutter arm handle length
- Deep gutter arm support
- Easy squeeze and release locking hand brakes
- Detachable folding backrest
- Padded vinyl seat
- Vinyl under seat storage
- PVC castors
- Conditional warranty applies

## SPECIFICATIONS

Gutter Arm Height:	103cm – 122cm
Seat Height:	54cm – 62cm
Seat Width:	38cm
Seating Space:	45.5cm
Seat Depth:	26.5cm
Overall Depth:	73cm (at highest setting)
Overall Width:	65cm
Between Gutter Arms:	39cm
Wheel Size:	20 x 2.5cm
Weight Capacity:	120kg
Net Weight:	8.6kg

# DUAL HEIGHT ADJUSTABLE Gutter Arm Rollator



# PA334 DUAL HEIGHT ADJUSTABLE GUTTER ARM ROLLATOR user guide



1. Length adjustable handles
2. Lockable handbrake lever
3. Gutter arm support pads
4. Handle length adjustment
5. Gutter height adjustment
6. Removable back rest
7. Gutter height adjustment
8. Lift up seat
9. Cane / umbrella holder. **9a** Top strap; **9b** Cup base
10. Storage bag
11. Front seat height adjustment
12. Rear seat height adjustment
13. Front wheel assembly
14. Rear wheel with brake



## ASSEMBLY

Unfold the frame with seat



## CASTORS

**Front:** Insert and slide the front wheel section (**13**) into the frame tubing equally at both ends), position to the desired height, locate the closest hole and lock in place with the C-Clips (**11b**) included.

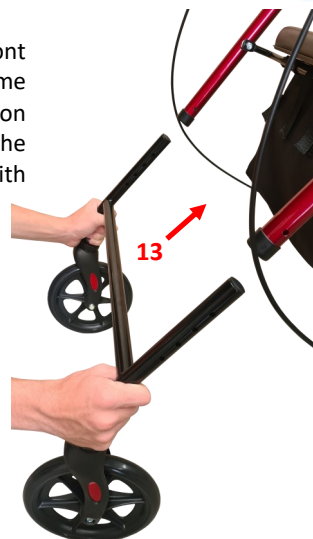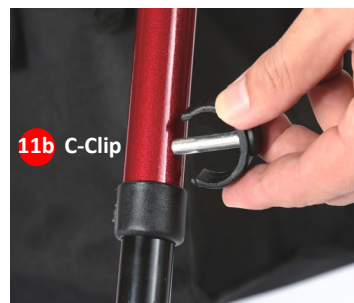

**Rear:** While pressing in the spring button (**12a**), insert and slide the rear wheel stem (**14**) into the frame tubing to the required height. Lock the spring button into the nearest hole. Repeat step for the other side to the same height position.



**Note:** Ensure all legs are at the same hole position for a level seating.

## GUTTER ARM RESTS

Insert the arm rest stem (**5**) into frame, position to the desired height. Locate the closest hole and lock into place with the knob bolt (**7a**)



## BRAKE & HANDLE

Insert handles (**1**) into gutter arm (**4**) and adjust to desired length and handle position. Tighten nut and flip lever (**4a**) to lock in place.

