

Before use, always check product is in good working condition.
Check regularly for wear and tear.

Ensure your weight is within the weight capacity of the unit and
the product is suitable for your needs and abilities.
Seek the advice of a suitably trained therapist or carer.

The rollator should be used with care and safety in mind.
This product is not a toy. Keep away from children.

Care must be taken to ensure stability:

- All wheels must be in contact with the ground.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Do not hang anything on the handlebars.

DO NOT USE if the rollator shows signs of damage, corrosion or rust

- Regularly check that all bolts and fasteners are secure
- Lubricate moving parts. A spray lubricant will also assist in water/moisture displacement and protect from corrosion.
- Clean using a slightly damp cloth and wipe dry.
- DO NOT use abrasive or solvent cleaning products.
- To prevent rust - Do not submerge the product in water and make sure to dry product completely before storing.
- Store in a dry place.

SPECIFICATIONS


Handle Height:	78cm – 90.5cm
Seat Height:	56cm
Seat & Walking Width:	45cm
Seat Depth:	23.5cm
Overall Width:	67cm
Overall Depth:	68cm (front castors inward)
Folded Dimensions	90 x 27 x 38cm
Wheel Size:	24 x 3.5cm (front) / 20 x 3.5cm (rear)
Weight Capacity:	136kg
Net Weight:	9kg






1. Height adjustable handles
2. Handbrake lever
3. Back rest
4. Cane holder strap (a) base (b)
5. Hooks for removable storage bag
6. Seat (6) and folding strap (6a)
7. Height adjustment
8. Folding elbow
9. Swivel front wheel
10. Rear wheel with brake system

UNFOLDING (OPENING) & FOLDING (CLOSING) FRAME

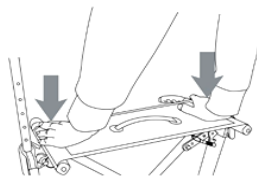

UNFOLD the rollator by pulling up the arms to an upright position; press down on sides of seat (6) until it clicks securely into position.

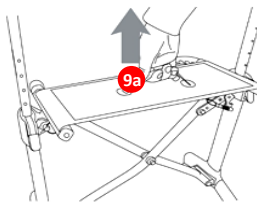


FOLD by pulling up strap on seat (6a) and press in sides of frame; as the sides close, it will release the elbows (8) allowing the handles to fold down to a compact size.

HEIGHT ADJUSTMENT



While pressing the red button (7), adjust the handle position to a suitable height; release button and lock to nearest hole. Repeat for other side.

HANDBRAKE & PARKING

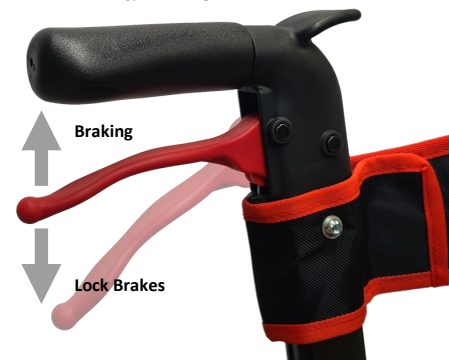

Gently squeeze brakes (2) upwards to reduce speed and to stop. Push brake handles down to activate park brake (a faint click will be heard when engaged). Squeeze brakes upwards to release.