

PAB811 / PAB821 EURO X-FOLD ROLLATOR user guide



PAB811

Handle Height:	81.5 – 91.5cm
Seat Width:	45cm
Seat Height:	48.0cm (~45cm when seated)
Seat Depth:	25cm
Overall Length:	72cm
Overall Width:	64cm
Between Handgrips:	45.5cm
Folded Width:	33cm
Wheel Size:	20 x 2.5cm
Weight Capacity:	150kg
Net Weight:	7.3kg

8" Front Castor



PAB821

Handle Height:	80.5 – 93cm
Seat Width:	45.5cm
Seat Height:	57cm
Seat Depth:	23cm
Overall Length:	73cm
Overall Width:	61.5cm
Between Handgrips:	46.7cm
Folded Width:	29.5cm
Front Wheel Size:	25 x 3.5cm
Rear Wheel Size:	20 x 3.5cm
Weight Capacity:	150kg
Net Weight:	7.5kg

10" Front Castor



PAB811 / PAB821 EURO X-FOLD ROLLATOR user guide



1. Height adjustable handles
2. Handbrake lever
3. Padded backrest
4. Padded seat
5. Handle height adjustment
6. Storage bag
7. Rear wheel with brake
8. Swivel front wheel

UNFOLDING THE ROLLATOR

Push down on the side bars of seat until it locks into position, a small click will be heard.

BRAKES

Caution: Always check brakes before using rollator.

Pull the brake lever (2) – up to brake and down to lock the brakes, pull upwards to disengage.

Note: Lock handbrakes when sitting down, while seated and getting up from the rollator.

Keep the brake handles free from dirt, dust and grime. Build-up of particles could cause your brakes to not engage properly. If the brakes are not engaging, or contacting the wheel while not being applied, adjustments can be made with the adjuster screw at the foot of the cable near the brake pad (tools may be required).

HANDLEBARS – HEIGHT ADJUSTMENT

Push button (5) in and raise / lower handle to the desired height and lock into place with closest suitable hole position. Repeat for other side.

FOLDING THE ROLLATOR

Pull up the strap handle in the centre of seat (4) to release lock and push the rollator frame from the sides to close

Caution: Be careful not to get fingers or any objects caught in between the frame.

GENERAL CARE & MAINTENANCE

Before use, always check product is in good working condition. Check regularly for wear and tear

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities.

Seek the advice of a suitably trained therapist or mobility specialist.

Care must be taken to ensure stability:

- All castors must be in contact with floor.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up from seat.
- Backrest must be attached to walker prior to use.
- Do not hang objects on the handles or backrest – this could cause the rollator to tip over.

DO NOT use if the rollator shows signs of damage, corrosion or rust

- Regular service is advised.
- Periodically check that bolts and nuts are securely fastened.
- Clean using a slightly damp cloth and wipe dry.
- To prevent rust - Do not submerge the product in water; dry product completely before storing.
- Lubricate moving parts.
- A spray lubricant will assist in water/moisture displacement and protect from corrosion
- Store in a dry place.

