

280kg HEAVY DUTY ROLLATOR

Before use, always check product is in good working condition. Check regularly for wear and tear.

Ensure your weight is within the weight capacity of the unit and the product is suitable for your needs and abilities. Seek the advice of a suitably trained therapist or qualified mobility specialist.

The rollator should be used with care and safety in mind. This product is not a toy. Keep away from children.

Care must be taken to ensure stability:

- All wheels must be in contact with the ground.
- Avoid tilting or leaning over too far.
- DO NOT use as a wheelchair or to transport someone.
- Lock handbrakes when sitting, while seated and getting up.
- Do not hang anything on the handlebars.

DO NOT USE if the walker shows signs of damage, corrosion or rust

- Regularly check that all bolts and fasteners are secure.
- Lubricate moving parts.
- Keep dry to prevent rust.
- Clean using a slightly damp cloth and wipe dry.
- DO NOT use abrasive or solvent cleaning products.
- Store in a dry place.

SPECIFICATIONS

Handle height:	89 – 102cm
Seat height:	55.5cm
Seat width:	48.5cm
Seating width:	54cm
Seat depth:	37cm
Backrest height:	31cm
Overall max height:	103cm
Overall max depth:	77cm (front castors inward)
Overall width:	73.5cm
Width between handles:	50.5cm
Distance between rear legs:	56.5cm
Wheel size:	20 x 3.5cm PVC
Weight capacity:	280kg
Unit weight:	8kg



Conditional warranty applies.
Visit pquip.com.au for terms and conditions

PQ HEALTHCARE AUSTRALIA Pty Ltd t/a
PHARMAQUIP HEALTHCARE
1/7 Monte Khoury Drive, Loganholme, Qld 4129


PQUIP[®]
PHARMAQUIP HEALTHCARE



- | | |
|------------------------------------|---------------------------------|
| 1. Height adjustable handles | 6. Vinyl storage bag |
| 2. Locking handbrake lever | 7. Folding bracket |
| 3. Back rest | 8. Rear wheel with brake system |
| 4. Handle height adjustment button | 9. Front swivel castor wheel |
| 5. Seat | 10. Folding lever (under seat) |

HEIGHT ADJUSTMENT

Press and hold the height adjustment button (4), adjust to desired height and button fully engages into the closest suitable hole. Repeat for other side to the same height position.

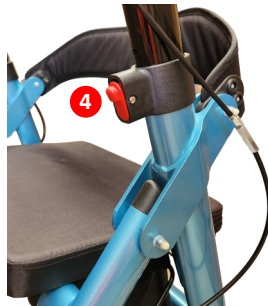

HANDBRAKE & PARKING



Braking: Squeeze brakes (2) gently (towards handles) to reduce speed and to stop.

Parking: Push brake handles downwards to activate park brake (a faint click will be heard when engaged). Squeeze brakes (towards handles) to release lock.

IMPORTANT:

- If the walker's rear wheels still roll while applying the brakes, DO NOT use it until the brakes have been checked and properly adjusted.
- Brakes must be locked before sitting, while seated, and when rising up off the seat.
TIP: Lean the front castors (backrest side) against a solid vertical surface (eg kerb or wall) to prevent the rollator from sliding back when standing up from the rollator.

UNFOLDING & FOLDING FRAME

Unfold the rollator by pulling the front and rear legs apart, make sure the folding bracket extends out fully.

Fold by raising the seat, pulling up the metal bar (10) underneath, and close frame.



PQ HEALTHCARE AUSTRALIA Pty Ltd t/a
PHARMAQUIP HEALTHCARE
1/7 Monte Khoury Drive, Loganholme, Qld 4129

PQUIP[®]
PHARMAQUIP HEALTHCARE